BG

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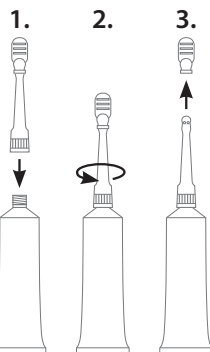
EN

Apimed® Hemopro ointment

FOR EASING THE DISCOMFORT CAUSED BY THE HEMORRHOIDS

Medical device:

The effect with medical device, as opposed to drugs, is primarily physical.

Ingredients:

PETROLATUM, LANOLIN CERA, ALCOHOL, AQUA, PROPOLIS CERA, ANTHEMIS NOBILIS

Intended use:

Apimed Hemopro ointment provides protective film for decreasing the rectal mucosa irritation and painful defecation.

Apimed Hemopro ointment creates protective film on the damaged rectum mucosa and thus facilitates the unpleasant symptoms caused by hemorrhoids (flatulence, pain, burning, itching), which are usually the result of secondary inflammation in the anal area. The resulting film prevents contact between the stool and damaged mucosa. With facilitated defecation (without pain and strain), mucosa recovers very quickly.

Instructions for use

During the acute periods, apply Apimed Hemopro ointment on clean anorectal area in the morning and in the evening. Later, when symptoms relieve apply only in the evening. It can also be applied by using applicator put on the tube instead of the cap. Before use, remove the protector from the applicator's top.

⚠️ Contraindications

Allergic reaction caused by hypersensitivity to an ingredient of the product.

Caution:

Persons allergic to propolis should not use this product!
Keep out of the reach of children

Safekeeping:

Store in a dry and cool place.

Net quantity:

20 g, REF 1101

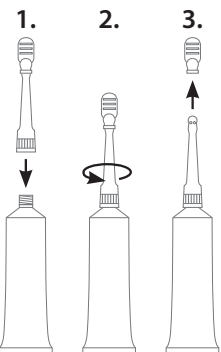
40 g, REF 1107

APIPHARMA d.o.o., Jeronima Kavanjina 26, 10.090 Zagreb, Croatia, EU

Batch number and Expiry date are printed on the package.

Application

Explanation of the symbols appearing:

	Consult instructions for use
	Expiry date
	Date of manufacture
	Warning
	Manufacturer
	Lot number
	Reference number